


Pregnancy-MS

Rapid Diagnostic Test for the detection
of hCG in human urine

One Step Test

MIDSTREAM TEST DEVICE

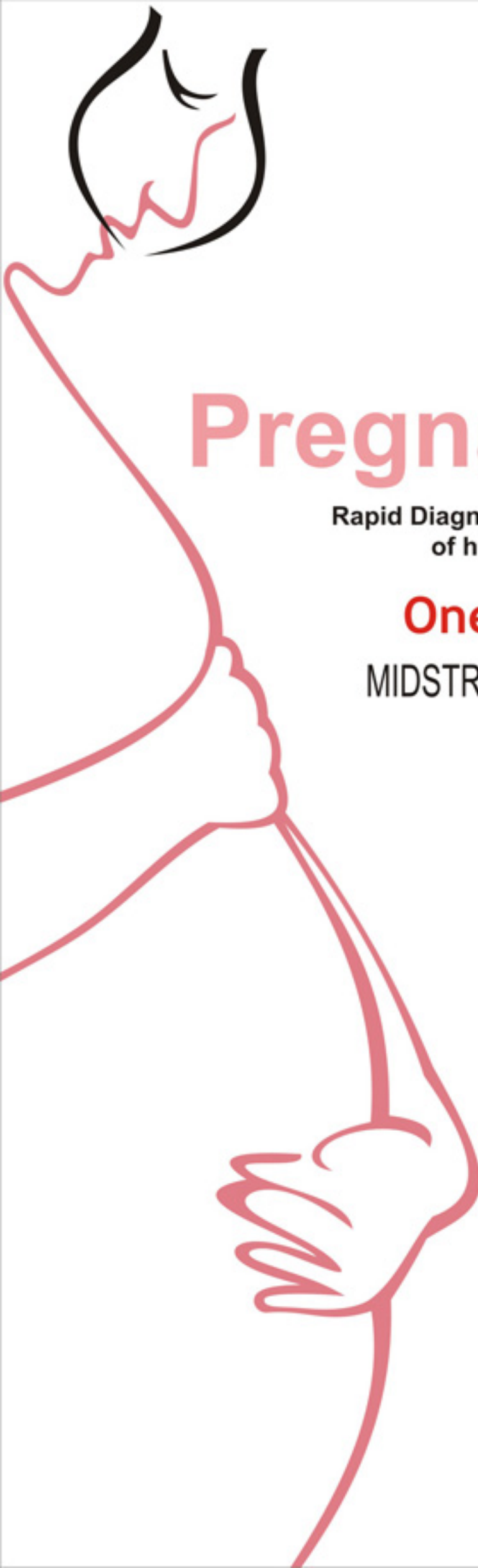

Pregnancy-MS

One Step , DO IT-YOURSELF PREGNANCY TEST

Mid Stream Test Device

Rapid, visual test for the detection of hCG in mid stream urine



- Detects hCG one day after the missed period
- Built in control for confidence reporting
- Fast test results : 5 minutes
- Clear demarcation between positive and negative
- Can be stored at 4°C to 30°C
- Shelf life : 2 years


TEST PROCEDURE AND INTERPRETATION OF TEST RESULTS

Test Procedure

1. Take off the protective cap covering the absorbent area of the test strip



2. Hold with the arrows pointing downwards, position the white absorbent area of the device in the urine stream for 5-10 seconds, so that the white absorbent area is wet

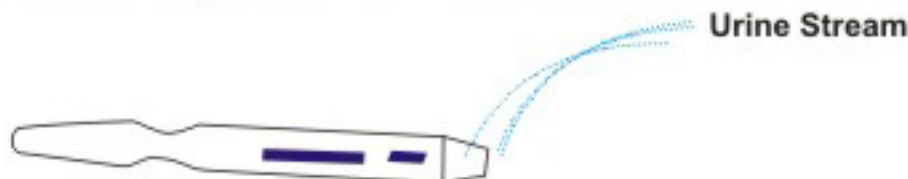

Place the device over the horizontal Surface. Read the results after 5 minutes

Interpretation of Test Results

Negative : Only one Pink- Purple colored band appear in the control region 'C'.

Positive : In addition to control band, one Pink- Purple colored band appears in test region 'T'.



Invalid : If neither the control nor the test band appears . Repeat the test with a new test device.

Performance characteristics :

- Detection limit** : 10mIU/ ml
- Sensitivity** : 100%
- Specificity** : 100%
- Pack Size** : One cassette per pack



Manufactured by
SSA DIAGNOSTICS & BIOTECH SYSTEMS
GOA (INDIA)
(Unit of **KEE GAD Biogen Pvt. Ltd.**)