



diagno**S**
t
i
c
k
S

Pregnancy

Rapid Diagnostic Test for the detection
of hCG in human urine

One Step dipstick Test

Pregnancy One step dipstick test



Sensitivity

Detects hCG one day after the missed period

Easy Interpretation of results

Clearly visible test and control band

Built in control band


For reliable and confident reporting

Rapid test results

Takes only 5 minutes


TEST PROCEDURE AND INTERPRETATION OF TEST RESULTS

Test procedure

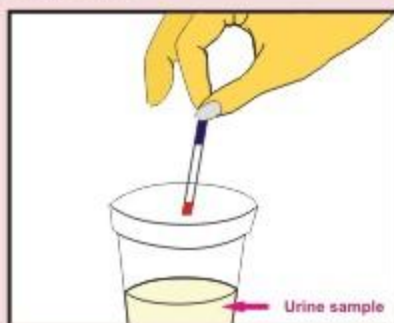

Place the dipstick in the urine sample submerging only the red area
Read the results after five minutes

Interpretation of test results

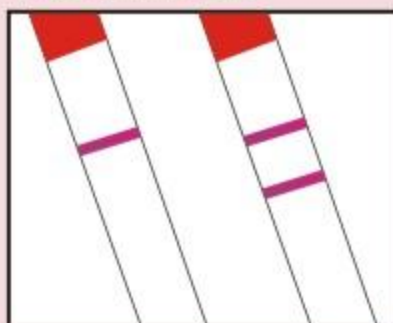

One pink-purple colored band on the dipstick-Negative
Two pink-purple colored bands on the dipstick-Positive

Invalid : If no color band appears on the dipstick. Repeat the test with the new dipstick.

Storage Temperature : 4°C to 30°C

Shelf life : 2 Years

Performance characteristics :

Detection limit : 10 mIU/ ml

Sensitivity : 99.7%

Specificity : 99.6%

Pack size : 25 dipsticks



Manufactured by
SSA DIAGNOSTICS & BIOTECH SYSTEMS
GOA (INDIA)
(Unit of **KEE GAD Biogen Pvt. Ltd.**)